Harvard Implant

Semi-permanent

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Harvard Implant

Semi-permanent



Dual cure semi-permanent luting cement for cementation of implant-based crowns and bridges

- Optimal bond strength, best compromise between temporary and permanent cementation
- Removal of restoration possible without damage
- Easy removable excess
- Elastic polymer film to minimize chewing pressure on the bone
- Excellent sealing
- No shrinkage, controlled slight expansion
- Optimal film thickness
- Very easy and comfortable to use
- Eugenol free


Physical Properties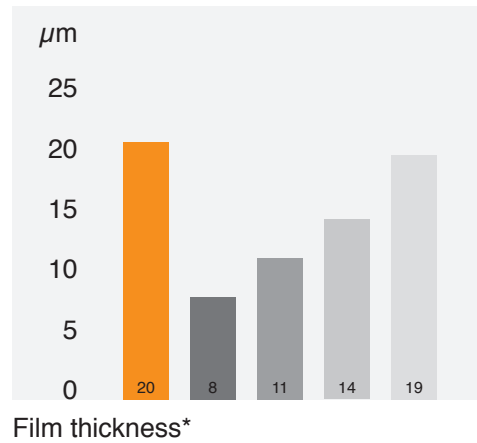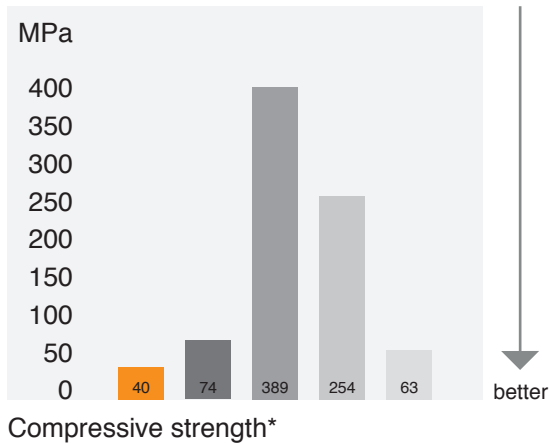

Low compressive strength = Higher elasticity!

- Harvard Implant
- implantlink semi (Detax)
- Premier Implant (Premier Dental)
- Multilink Implant (Ivoclar Vivadent)
- Systemp.link (Ivoclar Vivadent)

No shrinkage + controlled slight expansion = excellent sealing AND safe fixation!

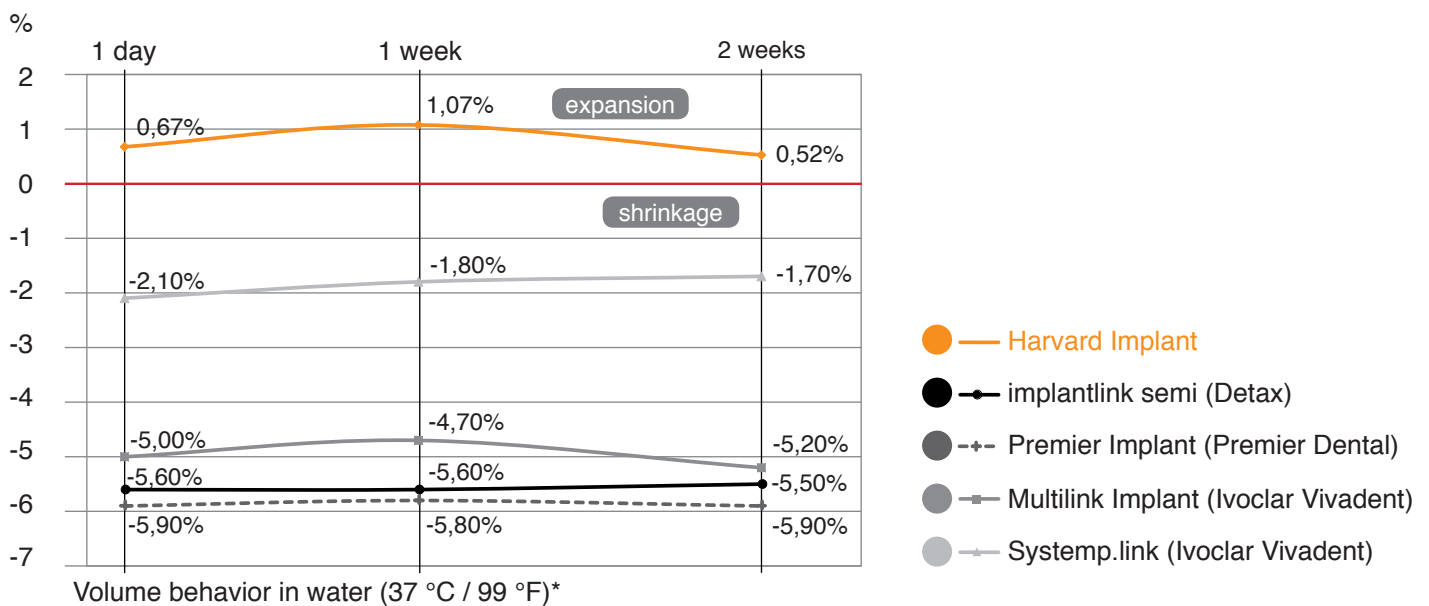* Internal data by Harvard Dental International

Properties and Advantages!

Based on Methacrylates, contains zinc oxide

Automatic mixing

- *Reliable and convenient*

Optimal bond strength between temporary and permanent cementation

- *Removal of restoration without damage*

No shrinkage

No entry of bacteria

- *Excellent sealing and safe fixation of the restoration*

Optimal film thickness

- *Excellent fit*

Dual cure

- *Immediate fixation*

- *Easy removal of material excess after light cure*

Elastic polymer film

- *Minimizes chewing pressure on the bone*

Eugenol free

- *Setting of composites is not inhibited*

- *Subsequent adhesive fixation is possible*

Good esthetics

Application

1. Abutments of Implants
2. Filling the bridge with Harvard Implant
3. Light cure
4. Harvard Implant excess material
5. Easy removal of excess material
6. Implants with the supraconstruction

Technical Data

Dual cure semi-permanent luting cement for cementation of implant-based crowns and bridges

Compressive strength	40 MPa
Film thickness	20 μm
Working time (23 °C / 73 °F)	2:00 min
Setting time (self cure)	approx. 4:00 min
Light cure	40 sec per area

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Dual-curing, Semi-permanent luting cement

Harvard Implant Semi-permanent is a dual cure Semi-permanent luting cement. Harvard Implant Semi-permanent is based on multifunctional methacrylic esters and contains zinc.

Harvard Implant Semi-permanent is suitable for Semi-permanent cementing of implant abutment retained crowns and bridges. It shows good elastic behavior and good seal thanks to controlled slight expansion. Harvard Implant Semi-permanent is delivered in 4:1 Minimix syringes, so it is very easy and comfortable to use.

Harvard Implant Semi-permanent does not influence the setting of resin-based filling and luting materials.

Contraindications

If the patient is known to be allergic to acrylates.

Side effects

In singular cases, Harvard Implant Semi-permanent may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

APPLICATION

Preparing the Minimix-Syringe

Harvard Dental International GmbH only recommends for Harvard Implant Semi-permanent the use of Harvard mixing cannulas Harvard Mini 4/10:1 S (Order.-No. 7093000).

Remove the cap of the Minimix-syringe and throw it away (do not use it again!). It is replaced by a 4:1 mixing cannula. Bleed the Minimix-syringe before applying the mixing cannula. Gently press the plunger until both components (base and catalyst) begin to flow out evenly. Make sure that the guidance of the Minimix-syringe is aligned with that of the mixing cannula and turn the cannula 90° clockwise until it locks in position. The material is now ready for application.

Note:

Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Store used syringe with fixed used mixing cannula in a dark place.

Cementing

Before cementing degrease, clean (e.g. with alcohol) and dry abutment and restoration.

Harvard Implant Semi-permanent is automatically mixed when dispensed with slight and even pressure. It can be applied directly into the restoration.

Apply a thin layer of Harvard Implant Semi-permanent to the inside of the restoration. Position the restoration within 2 minutes (working time at 23 °C) cautiously but firmly over the implant abutments.

For a fast fixation of the restoration cure Harvard Implant Semi-permanent at the marginal regions for 20 seconds per surface. In case of translucent materials it is also possible to expose to light through the material (40 seconds per surface). Without any light Harvard Implant Semi-permanent sets in the mouth within approx. **4 minutes**. After Harvard Implant Semi-permanent has set remove any excess.

Notes

If the restoration has to be removed for inspection, this must be done gently with slight and permanent traction. **Avoid jerky removal!** The remaining cement can be extracted almost in one piece. Use water spray and suction to remove any remaining residue.

Clean and disinfect abutment and restoration with a cotton pellet soaked in alcohol or another disinfectant. Rinse thoroughly with water and dry.

Now the restoration can be fixed again with Harvard Implant Semi-permanent (see Cementing).

Precautions

- Contact of resin pastes with skin, mucous membran and eyes should be avoided, especially by anyone having known allergies to acrylates.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- Commercially available medical gloves do not protect against the sensitizing effect of methacrylates.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of Harvard Implant Semi-permanent. Neither store the material in proximity of eugenol containing products, nor let it allow coming into contact with materials containing eugenol.

Storage information

Storage temperature 4-20 °C (39-68 °F). Protect from direct sunlight. Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Only for dental use!

Store product out of reach of children!

The instructions for use has to be kept for the duration of the application. The user is responsible for correct application of the product according to this instructions for use.

Order-No.	Article
7081400	5 ml Minimix-Spritze with 10 mixing cannulas 4:1 Shade universal
7093050	Harvard Mini 4/10:1 S-Brown Refill bag with 50 mixing cannulas

Harvard Dental International GmbH
Margaretenstr. 2 – 4
15366 Hoppegarten, Germany
Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de

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MIXPAC™ es una marca comercial registrada de Sulzer Mixpac AG Suiza.