HARVARD®

Harvard Bite

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Harvard Bite


Material for bite registration


- Outstanding grindability
- Precise and distortion-free registration
- Very fast setting
- Very low shrinkage
- High hardness
- Very good biocompatibility
- Easy to use in cartridge
- Pleasant vanilla aroma


Physical Properties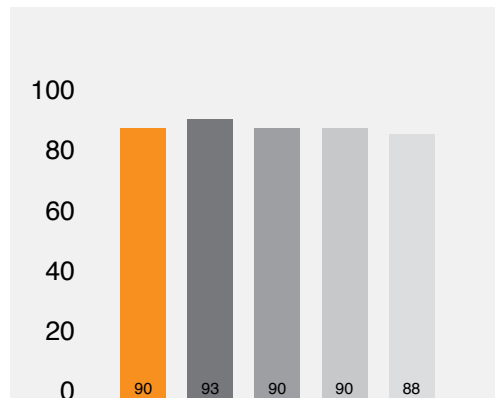

Shore A hardness after 24 Hours*

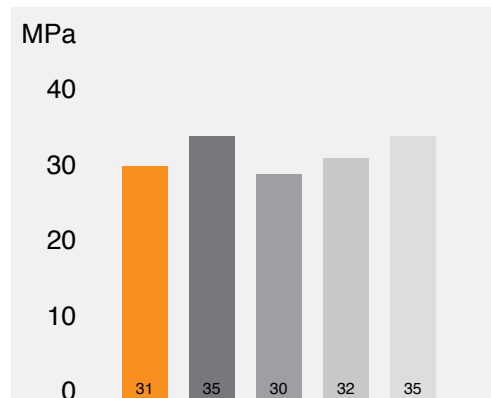
Shore D hardness after 24 hours*

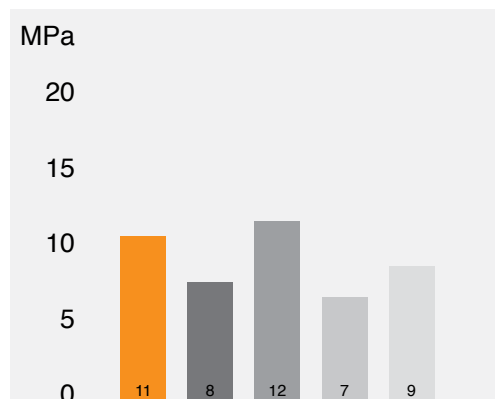Flexural strength*

■ Harvard Bite
 ■ Futar D (Kettenbach)
 ■ Aquasil Bite (Dentsply)
 ■ Jet Blue Bite Fast (Coltène Whaledent)
 ■ Imprint 4 Bite (3M ESPE)

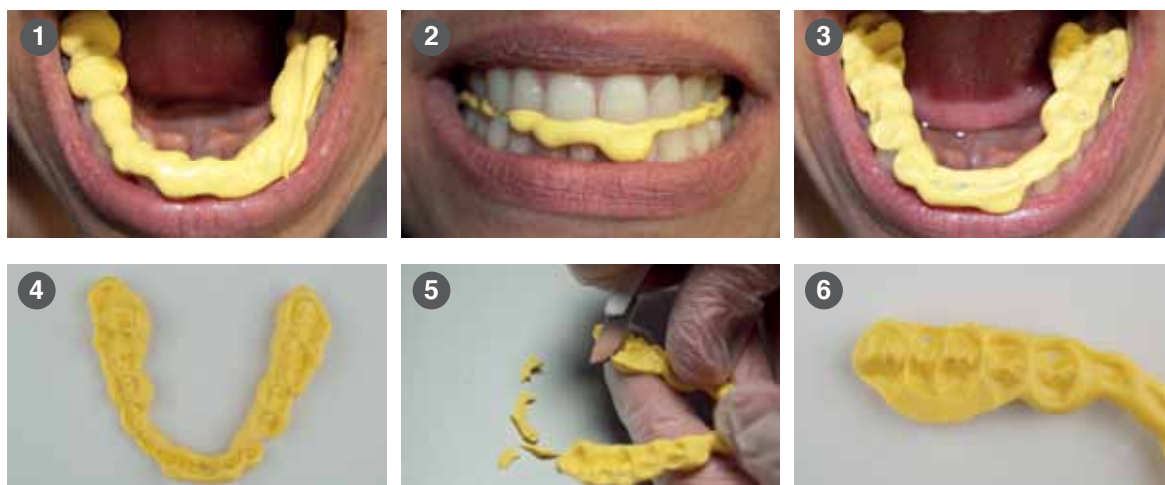1. Application of Harvard Bite on the teeth: Optimal stability
2. Precise and distortion-free registration
3. Intraoral bite registration: Outstanding reproduction of the bite situation
4. Bite registration
5. Excellent cuttability and grindability
6. After cutting

* Internal data by Harvard Dental International

Properties *and Advantages!*

Outstanding grindability

High hardness

- *Precise and distortion-free registration*
- *Easy to cut and mill, still extreme resistant*

Fast setting

- *Only 45 seconds setting-time in mouth*
- *Precise bite registration*

Optimal thixotropic flow

- *No flow from the tooth for a perfect registration*

Automix cartridge

- *Easy application out of the cartridge*

Application

Application

Registration

Bite registration

Harvard Bite has convinced me with its positive properties:

- The material displays a high level of recovery from deformation.
- It sets very fast, becomes very hard, yet remains flexible.
- It is well suitable with CMD patients.
- The material is very easy to cut.

The advantages of Harvard Bite especially in CMD patients are very good flow properties combined with a very high hardness after setting. This allows an ideal fixation of the registration.

I would recommend Harvard Bite to all dentists and colleagues at any time.

Kay Zischow, Zischow Dental Hamburg GmbH

Technical Data

Material for bite registration

Working time	> 20 sec
Setting time	45 sec
Strain in compression	0,5 %
Shore A hardness after 24 hours	90
Shore D hardness after 1 hour	30
Shore D hardness after 24 hours	31
Flexural strength	10 MPa
Elasticity modulus	28 MPa

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Harvard Bite

Bite registration material on A-silicone basis

Harvard Bite is a very hard and precise bite registration material with vanilla aroma based on addition curing silicones.

This biocompatible material provides an undistorted and high-precision bite registration with a short curing time. It can be applied quickly and bubble-free on the tooth rows with the cartridge.

Thixotropic characteristics and the stability associated with **Harvard Bite** prevents an inflow into the inter-dental area and allows for easy removal from the mouth.

Harvard Bite shows a high final hardness, is not brittle and may easily be cut and trimmed.

Indications

Bite registration material

Composition

Polysiloxanes; silicon dioxide, paraffin

Contraindications / side-effects

None known.

Application

1. Preparing the cartridge

Remove and discard the lock of the cartridge and mount the enclosed mixing cannula.

Equip the dispenser with the **Harvard Bite** cartridge.

Note:

Before every application, the first, approximately pea-sized material should be discarded in order to obtain the correct mix result. The used mixing cannula should be left on the cartridge as a cap until the next application.

2. Working time

Working time at 23°C is 20 seconds.

Recommended time in mouth is 45 seconds.

3. Bite registration

The mixed material is directly applied to the occlusal surfaces for registration.

The jaws should be closed after 20 seconds.

After a further 45 seconds, the material in the mouth has hardened, and the registration can be removed.

Important note: Avoid contact between the tooth areas or the bite registration material with latex gloves as the bonding reaction can be impaired by contamination.

Harvard Bite allows for an exact inspection of the fixation of the plaster model in the articulator. Registration made with **Harvard Bite** may be stored for many months without shrinking.

Important notices

Do not store **Harvard Bite** at temperatures above 25°C (78°F)!

Do not use the product after expiration date!

Prevent contamination with latex material!

Only intended for dental use!

Store product out of reach of children!

The instructions for use has to be kept for the duration of the application. The user is responsible for correct application of the product according to this instructions for use.

Order-No. Article

7083700 Harvard Bite 2 x 50 ml cartridge with 12 mixing cannulas
Colour yellow, vanilla aroma

7096000 Refill bags with 50 mixing cannulas

	„Harvard Dental International GmbH“ Margaretenstr. 2–4 15366 Hoppegarten	 Pagaminta Vokietijoje	Tel.: +49 (0) 30/99 28 978-0 Faks.: +49 (0) 30/99 28 978-19 info@harvard-dental-international.de www.harvard-dental-international.de	V01 2012-09 10287
--	---	--	---	----------------------