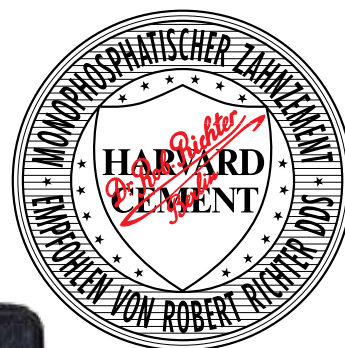
Harvard Cementas. Originalus nuo 1892.

**Harvard Cementas
OptiCaps®**

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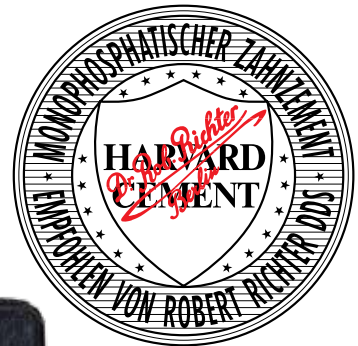
Harvard Cement. The Original. Since 1892.

**Harvard Cement
OptiCaps®**

**Harvard Cement
normal setting
fast setting**

**Harvard
Polycarboxylat
Cement**

Harvard Cement normal setting

Zinc phosphate cement for permanent luting of crowns and bridges and for lining

Properties

- High compressive strength
- No curing shrinkage
- Good biocompatibility
- Easy and safe application
- Low film thickness
- Unmatched price performance ratio

The classic!
Clinically proven
since 1892

Harvard Cement quick setting

Zinc phosphate cement with reduced setting time for permanent luting of crowns and bridges

Properties

- High compressive strength
- No curing shrinkage
- Good biocompatibility
- Easy and safe application
- Low film thickness
- Fast setting
- Unmatched price performance ratio

Short
setting
time!

Indications of Harvard Cement

Harvard Cement normal setting, Harvard Cement quick setting and Harvard Cement OptiCaps®:

- For permanent luting of crowns and bridges, inlays and onlays on natural core and for luting supraconstructions on implants.
For crowns, bridges, inlays and onlays made of:
 - zirconia
 - aluminium oxide
 - lithiumdisilicate
 - silicate
 - for conventional gold and non-precious metals

Harvard Cement normal setting additional qualified:

- For temporary fillings in posterior teeth
- As liner for all types of filling materials

Harvard Cement OptiCaps®

Zinc phosphate cement in capsules for permanent luting of crowns and bridges

Properties

- Consistent application with only 10 seconds mixing time
- Direct and precise application of creamy, Homogeneous cement on the restoration
- Avoids mistakes in mixing and dosing
- Sufficient for luting of 1 - 2 crowns from each capsule
- Working time of about 2 - 3 minutes

**Optimal
consistency
&
easy
Handling**

Harvard Polycarboxylat Cement

Polycarboxylate cement for permanent luting of crowns and bridges

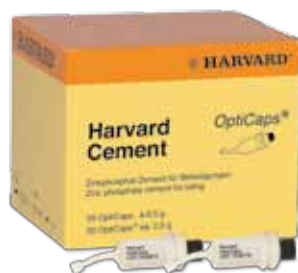
Properties

- Better biocompatibility than phosphate cements
- Non irritant for sensitive teeth
- Less irritant to the pulp than zinc phosphate cement
- Easy and safe application
- Unmatched price-performance ratio for luting materials

Indications of Harvard Polycarboxylat Cement

- For permanent luting of crowns and bridges, inlays and onlays on natural core and for luting supraconstructions on implants.
For crowns, bridges, inlays and onlays made of:
 - zirconia
 - aluminium oxide
 - lithiumdisilicate
 - for conventional gold and non precious metals
- For temporary fillings in posterior teeth
- As liner for all types of filling materials

Article

Order no.

Harvard Cement in OptiCaps®
shade yellowish white

Package with 10 OptiCaps® ea 0.5 g
Package with 50 OptiCaps® ea 0.5 g
Harvard Applier OptiCaps®

7081310
7081350
7092000

Harvard Cement normal and quick setting

Powder shade

- 1 - white
- 2 - bluish white
- 3 - yellowish white
- 4 - light yellow
- 5 - yellow
- 8 - pearl grey
- 12 - brown
- 15 - rose

Clinic powder 100 g
normal / quick
7002201 / 7001201
7002202 / 7001202
7002203 / 7001203
7002204 / 7001204
7002205 / 7001205
7002208 / 7001208
7002212 / 7001212
7002215 / 7001215

Single powder 35 g
normal / quick
7002501 / 7001501
7002502 / 7001502
7002503 / 7001503
7002504 / 7001504
7002505 / 7001505
7002508 / 7001508
7002512 / 7001512
7002515 / 7001515

Liquid

Clinic liquid 40 ml
normal / quick
7002300 / 7001300

Single liquid 15 ml
normal / quick
7002600 / 7001600

Harvard Polycarboxylat Cement

Powder shade

- 3 - yellowish white
- 4 - light yellow

Clinic powder 100 g
7031203
7031204

Single powder 35 g
7031503
7031504

Liquid

Clinic liquid 40 ml
7031300

Single liquid 15 ml
7031600

HARVARD CEMENT

normal härtend

Harvard Dental International GmbH
Margaretenstraße 2-4
15366 Hoppegarten
Germany
Tel +49 (0) 30 99 28 978-0
Fax +49 (0) 30 99 28 978-19
E-Mail: info@harvard-dental-international.de
www.harvard-dental-international.de

HARVARD CEMENT

normal setting
Zinc phosphate cement

Instruction for use for the dentist

Composition:

1 g powder contains:
860 mg zinc oxide, 86 mg magnesium oxide;
1 g liquid contains:
560 mg o-phosphoric acid.

Areas of Application:

Fixation of crowns and multiple bridges
- also on implants
- also for zirconium dioxide, aluminium oxide and lithium disilicate ceramics,
luting of inlays, cavity linings, temporary fillings in posterior teeth, reconstruction of the core, orthodontic works.

Dispensing Instructions:

Fixation consistency:
1.5 g powder : 1.0 g liquid
Cavity lining consistency:
2.1 g powder : 1.0 g liquid

Dispensing and testing climate for testing in accordance with DIN EN ISO 9917: Mix quantities indicated at 23 ± 1°C and a relative humidity of 50 ± 10%.

Processing Recommendations:

A clean, dry glass plate at ca. 20°C (68 °F) and a non-discolouring spatula are recommended for mixing.

Mixing:

In order to achieve a homogenous consistency, divide the measured quantity of powder into 4 portions (1/2 1/4 1/8 1/8). Beginning with the smallest quantity, stroke the powder into the measured liquid within 90 seconds. The filling consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste. The fixation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste. Discard remaining powder or impure liquid. Mixing Time: 90 seconds

Processing Time from start of mixing (at 23 °C, 72 °F):
For fixation: 3 minutes
For cavity lining: 2 minutes

Setting Time from end of mixing:

5:00 – 8:00 minutes

Special Information:

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
Irritations of the pulp may occur following the cementing and in the case of the mixture being used for cavity lining.
The cement must be applied to the dry tooth surface and must set without contact to moisture.
Close containers immediately after removing material.
- The liquid contains phosphoric acid and is corrosive. In the event of accidental eye contact, rinse immediately with large quantities of water.

Shades:

1 (white), 2 (bluish white), 3 (yellowish white), 4 (light yellow), 5 (yellow), 8 (pearl grey), 12 (brown), 15 (pink)

Package Sizes:

Powder: 100 g and 35 g
Liquid: 40 ml and 15 ml

HARVARD CEMENT normal setting

- Do not use after the expiration date.
- Storage temperature: 2–28 °C (36-82 °F).

Only for dental use!

Store product out of reach of children!

- Compliant with DIN EN ISO 9917. The instructions for use will be kept for the duration of the application. The user is responsible for correct application of the product according to this instructions for use.

Powder	Order no.
Clinic power 100 g	Single powder 35 g
1 - white	7002201
2 - bluish white	7002202
3 - yellowish white	7002203
4 - light yellow	7002204
5 - yellow	7002205
8 - pearl grey	7002208
12 - brown	7002212
15 - pink	7002215
Clinic liquid 40 ml	single liquid 15 ml
Transparent	7002300
	7002600

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